

VivaDiag™ SARS-CoV-2 Ag Saliva Rapid Test Self-test



VivaDiag™ SARS-CoV-2 Ag Saliva Rapid Test is for the rapid, qualitative detection of the nucleocapsid protein antigen from SARS-CoV-2 in human saliva. The test is for *in vitro* diagnostic use only. For self-testing use. It provides only an initial screening test result.

Specification

Test principle	Immunochemistry
Sensitivity	93.9%
Specificity	99.7%
Accuracy	97.7%
Sample type	Saliva
Sample volume	60µL
Test Time	15 minutes
Operation Temperature	15-30°C
Storage Temperature	2-30°C
Shelf life (unopened)	24 months

Features

- Fast results in 15 minutes
- Easy & user friendly
- Qualitative
- Reliable

How to Use

Specimen Collection




1. Spit the saliva




2. Draw the saliva

Detection

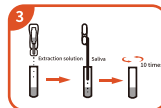

3. Prepare the specimen

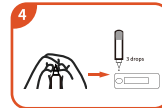

4. Apply the extracted specimen



5. Read the test result

Result Interpretation



Positive



Negative



Invalid

Approved by
**Medical Device Authority,
Ministry of Health Malaysia**



For instruction video, please scan the following QR code:



Malaysia Authorised Representative:



Covemed Sdn Bhd (1382675-W)

CT-09-01, Corporate Tower Subang Square, Jalan SS15/4G, 47500 Subang Jaya, Selangor, Malaysia
Tel: +603-8605 4677

Email: sales@covemed.com.my



Note: The VivaDiag™ SARS-CoV-2 Ag Saliva Rapid Test has ONLY been designed to act as a supplementary test for suspected cases of negative coronavirus nucleic acid detection or in conjunction with nucleic acid detection in the diagnosis of suspected cases. Results from nucleocapsid protein antigen testing should not be used as the sole basis to diagnose or exclude SARS-CoV-2 (COVID-19) infection or to inform infection status.

Not Available for Sales in the US

